SKÖTSELRÅD

MUNSKYDD

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CARE INSTRUCTIONS

FACE MASK

Face mask with Estrid Ericson's classic Elefant print. Wear it with caution, as a face mask does not protect you against infection. These face masks belong to category 2, which means that approximately 70 percent of the 3 micrometer particles can be stopped by the mask.

If the mask is used incorrectly, it can increase the risk of infection. It is therefore important that you follow these instructions carefully:

- Wash your hands with soap and water for at least 20 seconds each time before applying it, or clean your hands with hand sanitizer if you don't have access to soap and water.
- Make sure your mouth and nose are completely covered. You want to reduce air flowing between the mask and the face.
- Avoid touching the mask when you wear it. Try not to adjust it, and if you do, wash your hands first.
- If your mask feels very damp, if you drop it on the ground, or if it breaks, replace it immediately.
- Use the face mask for a maximum of 3 – 4 hours at a time.
- Remove your face mask using the ties, being careful not to touch the mask on the front.
- The face mask should be kept clean and washed in 90 degrees after each use.